NEWSEPT PLUS

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NEWSEPT PLUS

Newsept Plus[®] is a sterile normal saline solution in plastic containers.

Newsept Plus[®] as sterile, buffered, isotonic aqueous solution is indicated for rinsing of contact lenses of any type.

Each mL of solution contains 9 mg sodium chloride and 0.001 mg polyhexamethylene biguinide hydrochloride (preservative) in water for injections.

The automatic production, where no human hand interferes, and the high quality of DEMO S.A. assure the absolute sterility of the product.

INSTRUCTIONS FOR USE

- Before use, make sure that the container is not pierced.
- Remove the safety ring. Once the safety ring is removed, screw the cap until the spike, inside the cap, pierces the top of the bottle and leave it there for a few minutes, so that the hole will be properly formed. The spike, inside the cap, assures the safe closure of the bottle, when the cap is placed at its place.
- Always wash and dry your hands before handling your lenses.
- Before putting on your contact lenses, rinse carefully both sides with Newsept Plus[®] to remove the storage solution.
- Every time you want to remove your contact lenses, rinse them with Newsept Plus[®] carefully on both sides and place them in the storage case with the storage solution your ophthalmologist or specialized optician has suggested.

WARNING

- Newsept Plus[®] is not for nasal rinsing.
- Newsept Plus[®] is not injectable.
- Store at a temperature below 30°C.
- Six months after opening the container, discard the remaining unused solution.
- Do not use the product after the expiry date, if the solution is cloudy not clear and in case a precipitate appears or if the safety ring is broken or missing.
- In case an irritation occurs during usage of the product, contact your doctor or pharmacist and stop using it.
- Keep Newsept Plus[®] out of reach of children.

Packaging: Newsept Plus[®] is available in plastic bottles of 100 mL, 250 mL and 500 mL.

The leaflet was last revised in 09/2018.