## NEWSEPT TOTAL

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## **NEWSEPT TOTAL**

### **Multi-purpose solution for contact lenses with chamomile dry extract**

(Patent No: EP2604298)

Please read the leaflet carefully because it contains important information.

It contains tetrasodium etidronate for the removal of protein deposits.

#### **COMPOSITION:**

Sterile and isotonic solution. It contains: chamomile dry extract, boric acid, edetate disodium, sodium borate, sodium chloride, polyhexamethylene biguanide hydrochloride, tetrasodium etidronate and poloxamine.

#### **INDICATIONS:**

Newsept Total<sup>®</sup> is used for moisturisation, disinfection (without use of heat), cleaning, protein removal, lubrication, rinsing and storage of all types of contact lenses, as recommended by your optician. It is suitable for daily use.

#### **INSTRUCTIONS:**

Wash your hands before touching the contact lenses. After each use the contact lenses must be rinsed and soaked in the solution in order to be cleaned and disinfected. DEMO S.A. is proposing the following daily process:

- Step 1: Rub for ten (10) seconds each side of the contact lenses with a few drops of Newsept Total<sup>®</sup>.
- Step 2: Rinse for five (5) seconds each side of the contact lenses with Newsept Total<sup>®</sup>.
- Step 3: Fill the contact lenses case with fresh Newsept Total<sup>®</sup> solution and allow the contact lenses to soak for at least four (4) hours. You should never use the same solution twice.

Always follow your optician's recommendation about supplementary products or procedures. If your optician recommends that you skip the rubbing step, follow steps 2 and 3, as described previously. If

deposits remain on your contact lens, you may rinse with Newsept Total<sup>®</sup>. After each use discard the remaining solution left on the lens case and rinse the lens case with fresh Newsept Total<sup>®</sup> solution. Leave the case open to dry in the air and then your case is ready for use.

**STORAGE:**

If the lens case remains closed then the contact lenses can be stored for up to one (1) month and be ready for use. If you intend to store the lenses for more than one (1) month then they must be cleaned and disinfected at least every thirty (30) days using fresh Newsept Total<sup>®</sup>.

**WEEKLY CARE:**

Daily use of Newsept Total<sup>®</sup> assures sufficient protein removal and it is not necessary to use any other protein removal product.

**WARNINGS - PRECAUTIONS:**

- Newsept Total<sup>®</sup> is a harmless product; however you should follow your optician's recommendations as in other solutions of the same use.
- Do not use Newsept Total<sup>®</sup> if you know you are allergic to any of its ingredient.
- Do not use Newsept Total<sup>®</sup> if the solution is not clear, or if it has precipitate, if the safety ring of the bottle is broken or missing and if the container itself is not intact.
- Always wash your hands before touching your contact lenses.
- After each use throw away the solution from the lens case.
- Six (6) months after opening the bottle, throw away the remaining solution.
- Avoid touching the container tip to any surface or your eye because this may cause contamination of the solution or eye injury.
- Consult your doctor if you need to follow medication for your eyes while using Newsept Total<sup>®</sup>.
- Do not use heat disinfection method with Newsept Total<sup>®</sup>.
- In case you use ampoules of Newsept Total<sup>®</sup> at your office or during traveling, after opening the ampoule discard the remaining solution.
- Do not use after the expiry date stated on the container and the box.
- Store below 30°C.
- Keep Newsept Total<sup>®</sup> away from children.

**ATTENTION:**

Stop using Newsept Total<sup>®</sup> and contact your optician or doctor immediately in case irritation or excessive tearing occurs, symptoms that persist or intensify.

**PACKAGING:**

Newsept Total<sup>®</sup> is available in plastic bottles of 100 mL, 250 mL and 500 mL. It is also available in plastic ampoules of 10 mL, ideal for use at your office or during travelling.

The leaflet was last revised in 09/2018.

Chamomile was chosen for Newsept Total<sup>®</sup> composition due to its soothing effect on irritated body surfaces and due to the relief it provides when used for eye inflammation e.g. stye.

The healing properties of chamomile have been known since antiquity, Hippocrates, the father of Medicine, considered it a miraculous herb to relieve symptoms of many diseases.

Despite its modern day widespread use, researchers continue to constantly discover new information about its beneficial properties in the human body.

Chamomile properties: Common chamomile has excellent antibacterial, anti-inflammatory and soothing qualities.