

Χλωριούχο Νάτριο 0,9%

Στείρο ισότονο διάλυμα έκπλυσης

Σύνθεση

Κάθε 100 mL διαλύματος περιέχουν 0,9 g χλωριούχου νατρίου σε ύδωρ για ενέσιμα.

Περιγραφή – Συσκευασία

Το στείρο διάλυμα έκπλυσης χλωριούχου νατρίου 0,9% είναι ένα διαυγές, άχρωμο υδατικό διάλυμα και περιέχεται σε φιάλες πολυαιθυλενίου των 500 mL και 1000 mL με ειδικό καπάκι twist-off.

Ενδείξεις

Χρησιμοποιείται για χειρουργικές, ουρολογικές, οφθαλμικές εκπλύσεις καθώς και για εκπλύσεις καθετήρων.

Ιδιαίτερες προφυλάξεις

- Το διάλυμα να μην χρησιμοποιείται εάν περιέχει ίζημα ή είναι θολό.
- Να μην γίνεται η χρήση του προϊόντος αν το μπουκάλι είναι ανοιγμένο ή κατεστραμμένο.
- Πριν χρησιμοποιηθεί πρέπει να γίνει απολύμανση του στομίου της φιάλης με οινόπνευμα 70°.
- Μετά το άνοιγμα του μπουκαλιού, το προϊόν που δεν έχει χρησιμοποιηθεί θα πρέπει να απορρίπτεται.
- Να μη χρησιμοποιείται ως ενέσιμο.

Ημερομηνία λήξης του προϊόντος

- Η ημερομηνία λήξης αναγράφεται στην ετικέτα του μπουκαλιού.
- Να μη χρησιμοποιηθεί το προϊόν μετά την ημερομηνία λήξης του.

Συνθήκες φύλαξης

Να φυλάσσεται σε θερμοκρασία μικρότερη των 25°C.

Το παρόν φύλλο οδηγιών χρήσης αναθεωρήθηκε για τελευταία φορά τον 09/2018.



DEMO ABEE
ΒΙΟΜΗΧΑΝΙΑ ΦΑΡΜΑΚΩΝ

21^ο χλμ. Εθνικής Οδού Αθηνών-Λαμίας, 145 68 Αθήνα,
τηλ.: 210-81 61 802, 210-81 61 824, fax: 210-81 61 587,
e-mail: info@demo.gr www.demo.gr



CE 0653



Sodium Chloride 0.9%

Sterile isotonic irrigation solution

Composition

100 mL of solution contain 0.9 g sodium chloride in water for injections.

Description – Packaging

Sterile irrigation solution of sodium chloride 0.9% is a clear, colorless aqueous solution and is packaged in plastic polyethylene bottles of 500 mL and 1000 mL with a special twist off cap.

Indications

The solution is indicated for surgery, urological, ophthalmic rinsing and rinsing of catheters.

Special warnings

- Do not use if the solution is cloudy and in case a precipitate appears.
- Do not use if the bottle is opened or damaged.
- Disinfect the spout of the bottle with ethanol 70° before use.
- Upon opening the bottle, discard the remaining unused solution.
- Non-injectable.

Expiration date of the product

- Expiration date is stated on the label of the bottle.
- Do not use the product after the expiry date.

Storage conditions

Store below 25°C.

The leaflet was last revised in 09/2018.



DEMO S.A.
PHARMACEUTICAL INDUSTRY

21st Km Nat. Road Athens-Lamia 145 68 Athens, Greece.
Tel.: +30 210 8161802, Fax: +30 210 8161587
e-mail: info@demo.gr www.demo.gr



CE 0653



38148330-4